

VENNER PNEUX™ SYSTEM RECOMMENDED CLEANERS AND DISINFECTANTS

VENNER PNEUX TSM™

1. CLEANERS

The cleaning solutions listed below are approved for use with the Venner PneuX TSM™. PneuX Life Systems Inc has attained the information from Material Safety Data Sheets (MSDS), Chemical Resistance Data Sheets, manufacturers' and third party websites. Please follow the use of these cleaning solutions in accordance with the manufacturer's instructions.

PneuX Life Systems Inc shall not be held liable for any damage to the Venner PneuX TSM™ resulting from any variations in the chemical composition of the cleaning products.

Name of Cleaner	Manufacturer	Name of Cleaner	Manufacturer
ASEPTI-WIPE II	Ecolab	DISINFECTING	CLOROX
BIGUACID PLUS	Antiseptica	HOSPEC pH NEUTRAL DETERGENT	Hospec

2. DISINFECTANTS

■ Compatibility with Isopropyl Alcohol (IPA)

IPA is commonly used as a disinfectant and by nature has a rapid evaporation rate as well as low residue formulation. The use of IPA should not cause any functional defects to the Venner PneuX TSM™.

The following disinfectants have relatively high IPA content. It is recommended to conduct appropriate tests due to substantial or unknown levels of IPA. Please follow the use of these disinfectants in accordance with the manufacturer's instructions.

Name of Disinfectant	Manufacturer
ALCOPAD	CleanTex
GIGASEPT INSTRU AF	Schülke & Mayr GmbH
SUPER SANI-CLOTH	PDI

■ Compatibility with Chlorhexidine Solutions

Chlorhexidine is a disinfectant and antiseptic that is used for skin disinfection before surgery and to sterilise surgical instruments. The following disinfectants are brand names of Chlorhexidine. It is recommended to conduct appropriate tests before using with the Venner™ tracheal Seal Monitor. Please follow the use of these disinfectants in accordance with the manufacturer's instructions.

Manufacturer	Manufacturer	Manufacturer
Avagard	Corsodyl	Peridex
Betasept	Dyna-Hex	PerioGard
BactoShield CHG	Hibiclens	PerioChip
Chlorostat	Hibistat	Riohex
ChloraPrep	Operand Chlorhexidine Gluconate	XEPI Rub CET

VENNER PNEUX™ SYSTEM RECOMMENDED CLEANERS AND DISINFECTANTS

VENNER PNEUX™ ETT, TT and Extension Tube

The Venner PneuX™ ETT, TT and Extension Tube are single use devices which should be disposed of after each use, in accordance with standard departmental practices. Any attempts to reuse these devices pose a serious risk of cross-infection, reduces product reliability and functionality.

Notes:

Venner PneuX TSM™ was previously known as Venner™ Tracheal Seal Monitor

Venner™ PneuX™ ETT and TT was previously known as Venner™ PneuX™ P.Y.™ Endotracheal Tube and Tracheostomy Tube