



The Venner WireSafe™ is a single use box for preventing guidewire retentions. It is designed to be pre-filled and locked with appropriate equipment for the safe completion of Central Venous Catheter (CVC) or similar procedures.

Clinicians can only access the locked equipment in a Venner WireSafe™, by removing the guidewire from the patient, and using the guidewire as a key to open the locked pack. Follow these instructions to access a locked Venner WireSafe™.


- 1 Remove guidewire from the patient's catheter as per normal practice. Stabilise Venner WireSafe™ using index finger by covering the round magnet. Using other hand, insert the guidewire into the lightbulb shaped channel.

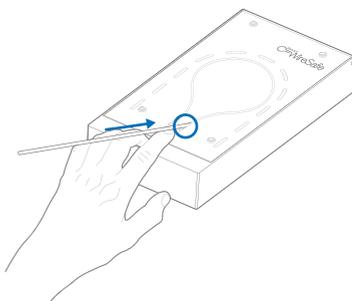

- 2 Push the guidewire through the channel until both ends of the guidewire are protruding outside of the channels.

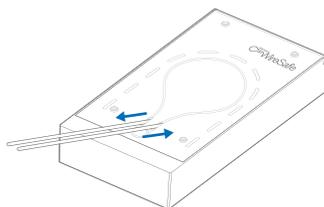


- 3 Move fingers from magnet onto the shelf at the front of the Venner WireSafe™. Whilst pushing down on the shelf with the fingers of one hand, hold the ends of the guidewire with the other hand and lift upwards to open the Venner WireSafe™.

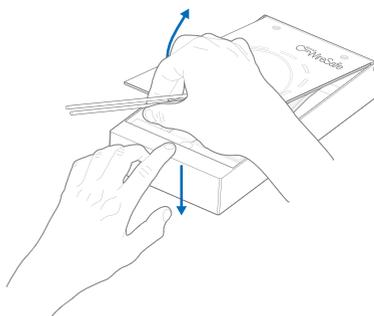


4

Use contents as per your normal practice to complete the procedure.

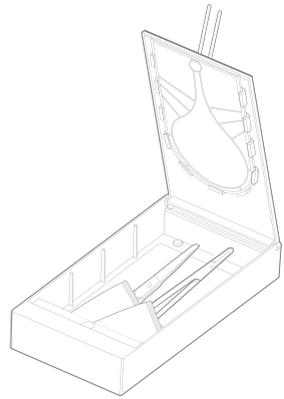

5

Dispose of Venner WireSafe™ in accordance with hospital policy.

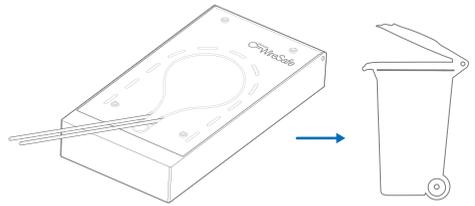

Venner Medical Technologies SA is the legal manufacturer of the Venner WireSafe™ box only. For assistance on the use of medical device(s) packed in the Venner WireSafe™, please follow instructions in accordance with the individual manufacturer.



Venner Medical Technologies SA
+44 (0)1534 872555
info@venner.com
www.vennermedical.com/venner-wiresafe



Venner Medical (Singapore) Pte Ltd
35 Joo Koon Circle,
Singapore 629110
www.vennermedical.com



Copyright© 2019 Venner Medical Technologies SA. All rights reserved. Venner and WireSafe are trademarks of the Venner Group of Companies. This product is covered by granted and pending patents of the Venner Group of Companies in the US and other territories. Visit: www.vennermedical.com/intellectual-property

VMT-WS-MK-003-01 REV G_EN